

TABS® Sidekicks

Mobility Monitor



Standard Features

- ▲ Restraint-Free
- ▲ Maintain dignity and privacy
- ▲ Sense movement to help prevent falls
- ▲ 90-day limited warranty

Specifications

Control Unit

- ▲ TABS® Select Monitor
- ▲ TABS® Professional Monitor
- ▲ Power Source: 9V Alkaline Battery

Chair model

- ▲ Dimensions: 13.4" x 11.6" x 1.2"

Bed model

- ▲ Dimensions: 26.1" x 4.1" x .2"

TABS® Sidekicks Model #26000 and #26100

Two Great Ways to Reduce the Risk of Falls

Our TABS® Professional and Select monitors act as the control unit for the TABS® Sidekicks pressure pad system. All the great features of TABS® Select and TABS® Professional monitors, such as multiple alarm tones and voice message alerts, can be used with the pressure pad.

This unique system gives you the flexibility to select the monitoring method that works best for each of your residents. Choose between the easy cord and clip assembly, or discreet pressure pads - or use them together - when you need to know when the resident is leaving a bed, chair or wheelchair.

The chair model can be placed on a wheelchair or other seat. The bed model is placed on top of a mattress. The pad is then connected to the TABS® monitor and the resident is situated comfortably on top of the pad. When the resident moves from the pad, the TABS® monitor alarm sounds either with a local alarm or at the nurse call or both.

TABS® Sidekicks pressure pads and industry leading TABS® mobility monitors combine to make the most versatile and cost effective fall monitoring system available. Try a chair or bed sensor pad today.

NOTE: TABS® Sidekicks pressure pads and other personal use products are not returnable.