

WanderGuard®

With the WanderGuard® system, you can choose from two resident-centered care alternatives: a 90-day device that complies with regular resident assessments, and a 12-month device, intended for the resident who presents a longer term departure risk.*



Standard Features

- 90-day and 12-month devices
- Waterproof case
- Small and lightweight
- Durable nylon-reinforced band
- Attractive, sleek design

WanderGuard Signaling Device Selection Chart

<i>WanderGuard 90-day signaling device</i>			
Signaling device model	Case color	Compatible systems	Non-compatible systems
36010	Beige with Peach back	ID, E, D, 202	
32013	Beige all over	E Plus, (D, 202)**	(D, 202)
34012	Beige with black back	FirstQ®	E, D, 202
<i>WanderGuard 1-year signaling device</i>			
Signaling device model	Case color	Compatible systems	Non-compatible systems
32122	Beige all over	E Plus, (D, 202)**	(D, 202)
35010	Beige with black back	FirstQ®	E Plus, D, 202

*43 percent of nursing facility residents lived in a facility for less than a year. Facts and Trends: The Nursing Facility Sourcebook, 1999.

** WanderGuard signaling devices (Models 32013 and 32122) may be used with WanderGuard D and 202 systems, but only if these systems have been tested and/or retuned for proper operation using WanderGuard E Series signaling devices. See retuning instructions in your Installation Manual or contact WanderGuard Technical Service for more information.

All devices should be placed on the resident's dominant wrist. When signaling device models 32013 or 32122 are used exclusively with the WanderGuard E Series departure alert system, the devices can be worn on the resident's wrist or ankle.

Resident-Centered Care with the 90-Day Signaling Device

Benefits of the 90-day signaling device:

- Least expensive signaling device on the market
- Encourages regular resident assessment in compliance with OBRA regulations
- Designed for one-time use to promote hygiene and infection control

According to ITT Hartford Insurance Group, 45% of all large loss wandering liability claims are a result of incidents that occur within the first 48 hours after admission. To reduce the risk of liability, the WanderGuard program suggests monitoring each resident with a 90-day device until the actual risk can be clearly determined.