

Basic-Check®

User Manual



REDUCING THE RISK OF FALLS THROUGH
INNOVATIVE TECHNOLOGY SINCE 1978
1-800-523-7956
www.bedcheck.com

How the Basic-Check® System Can Work for You

The Basic-Check® system consists of a control unit and a disposable pressure sensitive Sensormat®. When a patient's weight is removed from the Sensormat®, the control unit activates an alarm.

How to use the Basic-Check® System

⚠ WARNING ⚠

This monitor may not be compatible with mats manufactured by companies other than Bed-Check®. Use of incompatible mats may result in injury or death.

1. Plug the Sensormat® connector into the top of the control unit.
2. Place the Sensormat® so that it will be beneath the patient's buttocks.
3. The control unit will automatically arm itself when weight is placed on the Sensormat®.
4. If the patient lifts his weight off the Sensormat®, the Basic-Check® will immediately alarm.

5. To silence the alarm, simply press the red button on top of the control unit.
6. Test the system daily for effective operation.
7. System may not be effective for patients less than 100lb. Assure proper operation before putting into service.
8. Do not position either the unit or the Sensormat® in such a manner that causes one to pull on the other, causing stress to the connector wire or plug.

Care and Maintenance

The control unit can be wiped with a damp cloth or sponge using mild detergents or sanitizing solutions. Never use alcohol or strong chemical cleaners.

⚠ CAUTION ⚠

Never immerse the control unit in any liquid. Avoid placing the control unit in any area where it might come in contact with body fluids. Damage by fluids voids the warranty.

Internal maintenance should be performed by qualified personnel only.

Bending or folding the Sensormat® may cause loss of effective operation.

Battery

NOTE: The Basic-Check® control unit includes an alkaline battery. A low battery is indicated by a low constant beeping, even while the patient is on the Sensormat®. **DO NOT ATTEMPT TO RECHARGE THE BATTERY.**

LIMITED WARRANTY BASIC-CHECK® CONTROL UNIT AND SENSORMAT®

1. WARRANTIES: This limited Warranty is given by Bed-Check®, 1620 N 20th Cir, Lincoln, NE 68503.

2. DURATION: Basic-Check® Control Unit: begins on date delivered to purchaser and continues for a period of 2 years. Sensormat®: begins on date delivered to purchaser and continues for the lesser of 1 year or as specified on the Sensormat® label from the installation date.

3. TO WHOM THIS WARRANTY IS GIVEN: The limited warranty is given to the original purchaser of Bed-Check® products only.

4. WHAT IS COVERED: Defects in material and workmanship. Warrantor makes no other warranties express or implied, including merchantability or fitness of product for a particular purpose.

5. WHAT IS NOT COVERED: A) Any incidental, indirect or consequential loss, damage, or expense that may result from any defect, failure, or malfunction. B) Any incidental, indirect or consequential loss, damage, or expense that may result from use of product with another manufacturer's product. NOTE: Some states do not allow the exclusion or limitation of incidental or consequential damages. C) Any failure that may result from an accident, purchaser's abuse, neglect or failure to operate the product in accordance with the instructions provided herein. D) Any product which has the serial number altered, defaced or removed. E) Any product which has been altered or modified without the written consent of Bed-Check®.

6. RESPONSIBILITIES OF WARRANTOR: A) In the event of a defect, malfunction, or other failure not caused by misuse or damage, the warrantor will remedy the failure or defect without charge within a reasonable period of time. The remedy will consist of repair, replacement, or refund at the warrantor's option. If the product is no longer available, warrantor will supply a comparable product or refund the purchase price, at his option.

B) If this product or one of its component parts contains a defect or malfunc-

tion, after a reasonable number of attempts by the warrantor to remedy the defects or malfunctions, the purchaser will be entitled to either a refund or replacement of the product or its component part or parts. Replacement of a component part includes free installation.

7. RESPONSIBILITIES OF THE PURCHASER: A) Disinfect the product, if necessary, so that it is free of infectious matter. B) Ship the Sensormat® unfolded. C) Use the product with reasonable care. This limited warranty gives you specific legal rights. You may also have other rights which vary from state to state.

Please call Bed-Check® immediately if you experience problems.



1-800-523-7956

Mailing Address: PO Box 80238
Lincoln, NE 68501-0238
Shipping Address: 1620 N 20th Cir
Lincoln, NE 68503-1103



© 2008 The Stanley Works All Rights Reserved
Bed-Check, Basic-Check, and Sensormat are registered trademarks of Stanley Logistics, Inc.