




# Bed-Check® Classic-Check®

## Quick Programming Guide

This guide will assist you with manual programming of the Bed-Check® Classic-Check® control unit. More detailed manual programming instructions can be found in the User's Guide booklet.

For programming, only the control unit and power cord are needed. (Neither a Sensormat® nor the nurse call cord need to be plugged in during programming.)

### Arming/Disarming System

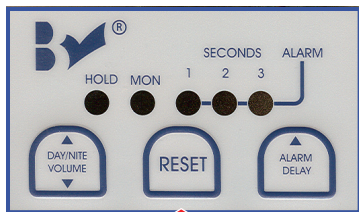


↑ Press and release

Press and release the RESET button to go from MONITOR to HOLD, or from HOLD to MONITOR, or to silence an alarm. Please note that if the system is on HOLD, it will reset itself and resume monitoring if weight is placed on the Sensormat®. In addition, if the RESET button is pressed to put on HOLD and weight remains on the Sensormat®, the unit will return to monitoring mode in 25 seconds.


**SETTINGS - Any of the following setting changes may be made independently of the others. We recommend setting tone before volume, as some tones may seem louder than others.**

### Day/Night Mode Settings

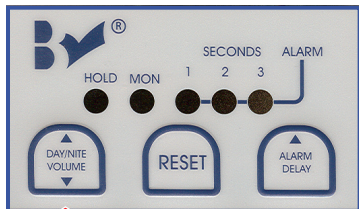


↑ Press and release

Press and release DAY/NITE button to switch between DAY and NITE modes. NITE mode is indicated by a blue light in the splash guard.

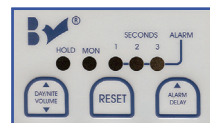


**DAY Mode**

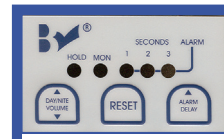

**NITE Mode**

### Alarm Delay Settings

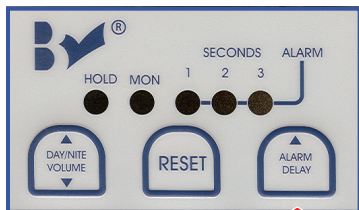

↑ Press and release

Press and release the ALARM DELAY button to adjust the alarm delay to 1, 2, or 3 seconds.



1 second



2 seconds



3 seconds

