

MODEL Vr IN-SERVICE REVIEW for BED Service

(Consult Model Vr ACCESS™ Program Guide and System User Guide)

PROPER INSTALLATION OF THE BED-CHECK® MODEL Vr® SYSTEM



A. Remove Sensormat® from plastic bag. Attach a rubber band and clip to each end of the Sensormat. This important step will keep the Sensormat positioned correctly on the mattress to ensure reliability. The Sensormat will work with either side facing up.



B. Make sure all sheets and covers are pulled down to the foot of the bed. Place Sensormat 3 to 5 inches below the bend in the mattress.



C. Attach the alligator clips to the fabric of the mattress, mattress piping or bed springs on both sides. You may now reposition sheets and covers. Record date Sensormat put in use on cord tag.



D. Plug the Sensormat, power supply cord and nurse call cord into their unique positions on the Model Vr® control unit.

IMPORTANT: Use only Sensormats manufactured by Bed-Check® with the Model Vr control unit.



E. Plug in the Model Vr power supply.

F. Plug in the nurse call cord into the facility's nurse call interface.
Note: An adapter may be required.



G. To secure the unit out of reach of the patient/resident, attach the Model Vr control unit to the bed frame* using the standard Velcro® strap, or to the headboard using an optional wire holster.

*Not directly under the mattress fold to avoid urine damage.



H. Help patient/resident into bed. The Model Vr unit will arm itself automatically. The yellow Hold lamp will blink several times and then the green Monitor lamp will light up. You can bypass the Hold delay and go directly to the Monitoring mode by pressing Reset once.



I. When the patient/resident attempts to exit the bed, an alarm will sound in the room and at the nurse station. Press the Reset button once to cancel alarm. Be sure to reset the nurse call station in the patient's/resident's room.



J. To remove the patient/resident from the bed, press the Reset button on the Model Vr one time. The unit will enter the temporary Hold mode for 25 seconds (Hold lamp will blink). If the patient leaves the bed during this time, the unit will transition to normal Hold mode (Hold lamp is lighted).



K. If the patient/resident has not been removed from the bed within 25 seconds, the control unit will enter the Monitor mode. Depressing the Reset button will cause the unit to once again enter the temporary Hold mode.



IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct position of Sensormat after any alarm.

Use Bed-Check products only to maintain Bed-Check warranty and to ensure System compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.



P.O. Box 80238, Lincoln, NE 68501 • 1-800-523-7956 • www.bedcheck.com