

Classic-Check® Models 72020 & 72021



Classic Features for Fall Risk Monitoring



Bed



Chair



Day and
Night Modes

With over 30 years of designing and manufacturing fall and restraint reduction monitoring systems, Bed-Check has learned what caregivers want most. Reliability. Simplicity. Versatility. Classic-Check has classic features for monitoring fall risk individuals.

- One-button Day/Night Mode control with 7 volume settings, programmable to different volumes for Day and Night.
- Ten programmable alarm tone settings: 6 tunes and 4 tones to easily distinguish fall alarms from other facility alarms.
- Night Light: soft, blue light within splash guard.
- Nurse call system compatibility.
- Variable delay settings: 1, 2, or 3 seconds (off Sensormat® pad until alarm).
- No on-off switch: monitors automatically with weight on the Sensormat pad, to help you meet Joint Commission requirements.
- Hold Mode: designed to enable removal of patient or resident by staff, without alarm sounding.
- Monitoring automatically restarts within 25 seconds when patient is on Sensormat pad even if the system has been placed in Hold mode.
- Convenient Velcro® mounting strap, or optional wire holster.
- Designed and tested to withstand high environmental static voltage.
- Integral 10 foot power and nurse call system cords.
- 110VAC powered: no batteries to replace.
- Every Classic-Check is assembled in the USA and 100% functionally tested before shipment. Two-year limited warranty.

Distributed by:

STANLEY®

800-523-7956 | Fax 402-475-4023 | www.bedcheck.com

Healthcare Solutions

Classic-Check® Models 72020 & 72021



PURPOSE

The Classic-Check control unit is intended to be used in conjunction with a Sensormat® pad as a system for the monitoring of fall risk patients or residents in a bed or stationary chair. Classic-Check provides an audible alarm to alert the caregiver when the person begins to get up unassisted. Classic-Check is nurse call system compatible, and comes with an integral ten foot nurse call system cord (standard) for connection if desired.

COMPONENTS OF THE SYSTEM

The System consists of the Classic-Check control unit with integral power and nurse call system cords, and a pressure-sensitive Sensormat pad (ordered separately) with a cord that is connected to the Classic-Check. Sensormat pads are available for bed and chair. See Options below for available holsters and hangers.

POWER REQUIREMENTS

The Classic-Check is powered by plugging the integral power supply into a standard wall outlet (110VAC). Power is converted to 12VDC before entering the Classic-Check case (no line voltage in the monitor).

INSTALLATION

The Classic-Check may be mounted on the bed headboard or frame, on the head wall, or the chair back using the standard Velcro® strap or optional wire holster (see Options). It should be inaccessible to the monitored person. The appropriate Sensormat pad is installed per its in-service instructions. See the Classic-Check User Guide for complete instructions.

IN-SERVICE

Ask your professional Bed-Check Distributor about in-service training options.

FEATURES AND OPERATION

Classic-Check has many standard features designed to facilitate the monitoring of fall risk patients.

- Monitoring begins automatically 5 seconds after patient weight is sensed on the Sensormat pad.
- Begin monitoring immediately by pushing RESET any time a patient is on the Sensormat pad.
- The alarm delay is 1, 2, or 3 seconds (settable) after weight is removed from the Sensormat pad. The Classic-Check sounds an audible alarm tone and signals the nurse call system.
- One touch operation starts a 25 second temporary hold period, in which the patient can be re-positioned or assisted off the Sensormat pad without causing the system to alarm.
- Day and Night modes. Setting to "Nite" illuminates the splash shield with a soft blue light and changes alarm volume to your pre-set "Nite" level.
- LEDs indicate HOLD and MONitor modes, and the 1, 2, or 3 second delay countdown.
- 7 volume settings each in Day and Night modes.
- 4 tone and 6 tune settings.
- 10 foot power and nurse call systems cords.
- ETL listed. Conforms to UL® Standard 1069 and to CSA Standard C22.2 No. 205R.



OPTIONS

- Classic-Check with Accessory Port at nurse call connection end of nurse call system cord. This permits the call button cord and the Classic-Check cord to share the same call station port without using a Y cord adapter. (Model 72021)
- Without Accessory Port (Model 72020)
- Holster: a wire basket holster is available with a 2" depth hanger. (Model 72044)

800-523-7956 | Fax 402-475-4023 | www.bedcheck.com

STANLEY®

Healthcare Solutions