

CHAIR-CHECK® II IN-SERVICE REVIEW

for CHAIR

PROPER INSTALLATION OF THE BED-CHECK® CHAIR-CHECK II SYSTEM ON A CHAIR

For more information, consult Chair-Check II User Guide



A. Remove Sensormat from plastic bag. Use the adhesive strips on the back of Sensormat to keep it properly positioned and to ensure reliability. Most often, place Sensormat in the seat of the chair 4 inches from the back of the chair, across the width of the seat.



B. Plug the Chair Sensormat phone-style jack into the Chair-Check II control unit.

IMPORTANT: Use only Sensormats manufactured by Bed-Check with the Chair-Check II control unit.



C. To secure the unit out of reach of the patient/resident, attach the Chair-Check II control unit to the back of the chair using either the standard metal hanger or an optional wire holster. Record date Sensormat put in use on cord tag.



D. Seat the patient in the chair. The Chair-Check II control unit will arm itself automatically. The alarm will beep one time, followed by a blinking red light located next to the Reset button.



E. When the patient/resident attempts to exit the chair or leans too far forward, the alarm will sound. Alarm requires immediate intervention. The unit will automatically resume monitoring with patient weight back on the Sensormat.



F. After alarm has sounded, caregiver should press the Reset button, silencing the alarm and allowing the Chair-Check II control unit to go into stand-by mode.



G. To remove the patient/resident from the chair, press the Reset button on the Chair-Check II one time and the Chair-Check II will go into stand-by mode.



H. If the patient/resident has not been removed from the chair in one minute, the Chair-Check II will resume monitoring. This will cause an audible beep and the red blinking light will resume.



Seat Belt Installation:

1. Lay Belt across seat.
2. Unscrew the two screws holding the back edge of the wheelchair seat in place.
3. Replace the screws, placing each screw through the metal eyelet at the end of the Belt.
4. Clip a monitor to the back of the wheelchair. With the Seat Belt buckle latched, connect the Seat Belt cable into the jack on the monitor. The monitor should emit a confirmation tone.



STANLEY



P.O. Box 80238, Lincoln, NE 68501

1-800-523-7956 • www.bedcheck.com