

COVID-19 QUICK DEPLOYMENT BUNDLE

Patient Call for Temporary Care Areas

AeroScout® RTLS users can add patient call functionality, including Call Stations/ Pendants and remote installation support



Challenges


Due to the recent COVID-19 pandemic, hospitals and health systems are establishing temporary care facilities to house the overflow of patients. We are seeing many hospitals setting up patient beds in non-traditional areas within existing hospitals, such as hallways and cafeterias. These facilities need a solution for patients in the temporary care areas to call for assistance.

Solution

STANLEY Healthcare's AeroScout Real Time Location System (RTLS) platform automatically tracks and monitors the location and status of assets, people, and resources throughout hospitals and healthcare facilities. Hospitals can leverage their existing AeroScout system to receive notifications from Wi-Fi Pendants and Wi-Fi Call Stations to enable patient call functionality for temporary patient care areas that have sufficient Wi-Fi infrastructure to support the system.

Packages include:

Customer's Existing RTLS Platform (MobileView®)

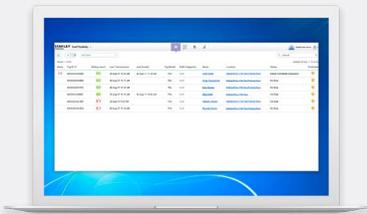

Wi-Fi Pendant Tag Kit

20 pendant tags with belt clips and accessories



Wi-Fi Call Station Kit

20 Wireless Call Stations and push button cords



COVID-19 RESPONSE LIMITED-TIME OFFER*

To support the need to set up temporary hospitals and patient care areas due to the COVID-19 pandemic, we have developed a special limited-time package that enables facilities to quickly deploy the Patient Emergency Call functionality through the Asset Management module. This special package allows you to quickly deploy Wi-Fi Call Stations and Wi-Fi Pendants to patient beds in temporary care areas of the hospital.



How It Works

- **Call Station/Pendants:** Wi-Fi Call Station or Wi-Fi Pendants Tags can be placed or attached to patient beds in temporarily care areas.
- **Monitor:** The location and status of Wi-Fi Call Stations/Pendants is visible on your existing RTLS system.
- **Find and Alert:** Patients push the button for help, which alerts caregivers via standard alerting channels that exist in MobileView such as Instant Notifier. Caregivers can reset the alarm from the pendant or call station where it originated.
- **Reports:** See alert volumes, response times and more.

Deployment

We have designed a simplified and remote implementation process that enables hospitals and healthcare facilities to quickly take advantage of our Patient Call on MobileView bundle. This enables you to be fully operational on an accelerated schedule.

CONTACT US

Contact your STANLEY Healthcare Client Executive with any questions or call for General Inquiries:

+1-888-622-6992

INTERNATIONAL: +44 (0)1753 500608

Assumptions and Considerations

- Offering is intended for existing STANLEY Healthcare customers with MobileView.
- Support services will be delivered remotely.
- Requires remote access to MobileView server for tag and event configuration.
- Call Stations and pendants, Call Alerts and Dismiss events need to be set up in MobileView.
- The bundle does not include Tuning Services and/or Location Optimization; location accuracy is dependent upon the characteristics of the customer's Wi-Fi network, including Access Point (AP) density and network design.
- Pendants and Call Stations will be pre-configured with IBSS, CCX or WDS based on the type of infrastructure the facility uses.
- Batteries for call stations and pendants provided.
- This offering is not UL listed for use in hospitals. Due to the temporary nature in a crisis scenario, buyer must confirm such standards and requirements are being waived.
- This offering is not a Nurse Call or Emergency Call system.

*DISCLAIMER

All product and pricing offerings are subject to a mutually-agreed quote and acceptance by STANLEY Healthcare of customer's purchase order. STANLEY Healthcare reserves the right to set criteria for this promotion based on factors including, but not limited to, contractual terms and product availability. The offer period begins on April 1, 2020 and expires on June 30, 2020. Customer must submit a purchase order to STANLEY Healthcare no later than June 30, 2020. This offer may not be applied retroactively for orders placed prior to April 1, 2020. STANLEY Healthcare reserves the right to modify or cancel this offer at any time without notice. Offer valid for customers only in the United States, Canada and other selected geographies.



About STANLEY Healthcare

Over 15,000 hospitals and senior living communities rely on STANLEY Healthcare solutions to empower caregivers to deliver better care. STANLEY Healthcare is a part of Stanley Black & Decker and a proud supporter of the Alzheimer's Association®. Learn more at stanleyhealthcare.com.