

COVID-19 QUICK DEPLOYMENT BUNDLE

Fall Management for Temporary Care Areas



Challenges

Due to the recent COVID-19 pandemic, hospitals and health systems are establishing temporary care facilities to house the overflow of patients. Many hospitals are setting up patient beds in non-traditional areas within existing hospitals, such as hallways and cafeterias, to deal with the influx of sick patients. These facilities also need a solution for patients in these temporary care areas to address essential care functions such as fall risk mitigation.

Solution

STANLEY Healthcare's UMP® Fall Management Solution is a self-deployable, user-friendly solution offering advanced fall functionality including multiple tones, volume, and delay settings. The monitor is powered by a standard 9V battery (included) and offers low-battery alerts for added safety. The two bundle options include:

MONITOR WITH BED PAD

UMP Sentry™ Fall Monitor Kit

Includes monitor & battery



UMP 45-Day Bed Pad



MONITOR WITH PULL-CORD

UMP Pull-Cord Monitor Kit

Includes monitor, pull string & battery



COVID-19 RESPONSE LIMITED-TIME OFFER*

To support the need to set up temporary hospitals and patient care areas due to the COVID-19 pandemic, we have developed a special limited-time package that enables facilities to quickly deploy the UMP Fall Management Solution. This special package allows you to quickly deploy in temporary care area of the hospital. This bundle includes components to cover 25, 50, or 100 patient beds. Additional monitor/pad bundles can be added to scale to meet needs.



Deployment

The UMP monitor and bed pad are easily set up, allowing hospitals and healthcare facilities to quickly self-install fall management equipment in temporary care areas. Place the battery into the UMP monitor. Plug in the included sensor pad to the bottom of the monitor. Follow the placement instructions on the sensor pad and lie the patient in the bed. The monitor will “beep” indicating that the patient has activated the system and is being monitored for exits. If the patient tries to exit and the monitor alarms, simultaneously push the two large buttons on the top of the monitor to cancel and silence the monitor.

Assumptions & Considerations

- Batteries for fall monitors are provided. Under normal use, battery life is six months.
- Single patient use: a new pad should be used for each patient to avoid contact pathogens.
- Test system before each use.
- Pads provided in bundle are for bed use only.

CONTACT US

Contact your STANLEY Healthcare Client Executive with any questions or call for General Inquiries:

+1-888-622-6992

*DISCLAIMER

All product and pricing offerings are subject to a mutually-agreed quote and acceptance by STANLEY Healthcare of customer's purchase order. STANLEY Healthcare reserves the right to set criteria for this promotion based on factors including, but not limited to, contractual terms and product availability. The offer period begins on April 1, 2020 and expires on June 30, 2020. Customer must submit a purchase order to STANLEY Healthcare no later than June 30, 2020. This offer may not be applied retroactively for orders placed prior to April 1, 2020. STANLEY Healthcare reserves the right to modify or cancel this offer at any time without notice. Offer valid for customers only in the United States and Canada only.



About STANLEY Healthcare

Over 15,000 hospitals and senior living communities rely on STANLEY Healthcare solutions to empower caregivers to deliver better care. STANLEY Healthcare is a part of Stanley Black & Decker and a proud supporter of the Alzheimer's Association®. Learn more at stanleyhealthcare.com.