

# M210 Fall Monitor

The M210 Fall Management System consists of pressure-sensitive Sensormat pads which detect the presence of a patient/resident, and a monitor which provides staff with alarm information.

The monitor provides Pad Exit alarms and Confirmation tones. All audible alarms can be cancelled by pressing the Reset Button on the front of the monitor.

If the patient/resident exits the bed or chair pad for longer than a pre-selected number of seconds (0, 1 or 2), the monitor issues visual and auditory alarms to alert staff that the patient/resident may need immediate assistance. For the floor pad, the alarm is sounded when pressure is added to the pad.

If a technical problem prevents the pad from monitoring (e.g., pad or Nurse Call cable disconnect, low battery condition, etc.), the monitor issues visual and Confirmation tones to notify staff.

If a Nurse Call system is connected, alarms can be sent to the Nurse Call station. The staff member, having been alerted that a patient/resident is exiting the bed or chair, must provide immediate assistance.

The monitor is latex-free and includes a bumper around the perimeter for added durability.

For easy set up, all configurations and Firmware updates are done over-the-air, wirelessly, using the Fall Management iOS mobile app.



## Monitor Specifications

Model Number	M210	
Battery Type:	CR123A Lithium 3V, Qty 3, replaceable	
Typical Life:	90 days battery life with typical usage.	
Low Battery:	Approx. 7 days warning	
Dimensions (W x H x D)	Approx. 8.5 x 14.8 x 4.0 cm (3.4 x 5.70 x 1.65 in)	
Weight	Approx. 275 g (9.70 oz) (with batteries) 175 g (6.17 oz) (without batteries)	
Operating Temp.	0 - 50°C (32 - 125°F)	
Radio	BLE 5.0 NFC	Used for configuration and Firmware updates.
Relative Humidity	5 - 90 % RH (non-condensing)	
Mounting	BMB-1000 - Wall Mounting Kit Removable Wheelchair Clip - objects up to 5/16" thick Large wire clip - objects from 5/16" to 2¼" thick	
Audible Indicators	"Low " Minimum Level ≈ 78 dBA - at 1 foot from monitor "Medium" Medium Level ≈ 85 dBA - at 1 foot from monitor "High" Maximum Level ≈ 96 dBA - at 1 foot from monitor	
LED Indicators	Reset Button: Green - Start-up, Arming/Armed, Button Press Red - Pad Alarm Amber - Hold/Suspend Mode Status Indicator Icon Green - Delay Settings Yellow- Low Battery	